


Application:

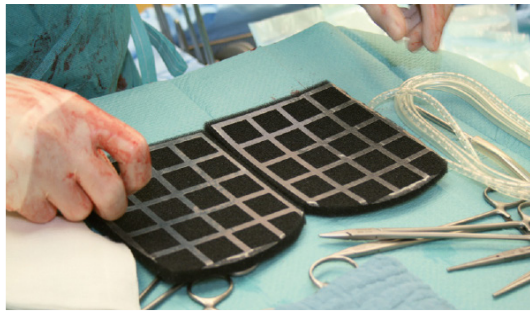

1. Adaptation of PLV ProVac wound foams with adhesive meshes to the respective findings and removal of protective films.



2. Positioning of the drain(s) on the adhesive mesh.



3. Folding of the foams and introduction of the set into the wound cavity. Lateral routing out of drain with help of trocar.



4. Closure of wound and sealing with film, hydrocolloid or foam dressings. Removal of trocar from drain.



5. Connection of drain to PLV universal adapter, opening of front side of universal adapter for connection of suction connector and then activation of vacuum.

Further treatment: The closed vacuum therapy system should be replaced after 3-5 days. Depending on the findings in the wound cavity, the treatment can be continued with fresh foam or the wound closed without foam.